

# CT-guided Lung Biopsy

Alexander Kupferthaler, MD, EBIR

Interventional Radiologist, Department of Diagnostic and Interventional Radiology,  
Ordensklinikum Linz Barmherzige Schwestern



## Case History

58-year-old male patient.

Diagnosis: Solitary plasma cell myeloma in 2018.

Recurrence in 2023 with a lung lesion in the right lung apex paravertebral.




Biopsy needed for treatment planning and genetic analysis.

## Procedure

- The patient was under general anesthesia with disconnection of the tube for breath-hold when scanning, needle puncture, and needle advancement.
- The lesion size was 1.6x1.9 cm, and the trajectory path to the target was around 14 cm.
- After automatic registration of the robot and planning, a 17G coaxial needle was advanced to the target. The correct position was confirmed with a single control scan at the target.
- Next, an 18G biopsy needle was advanced through the middle of the lesion, and the specimen was taken. The last step was to remove the coaxial needle. The biopsy yielded accurate sampling, and recurrence was confirmed.
- A non-significant pneumothorax, 5 mm, was visible on the postprocedural scan, but no additional treatment steps were required.

## Key Takeaways

- Micromate™ enabled the extraction of a biopsy from a challenging lesion – paravertebral through the narrow intercostal space.
- The robot's navigation and guidance features facilitated the planning and execution process.
- The system provided the clinical team with ease of use, accuracy, and the ability to perform challenging approaches.

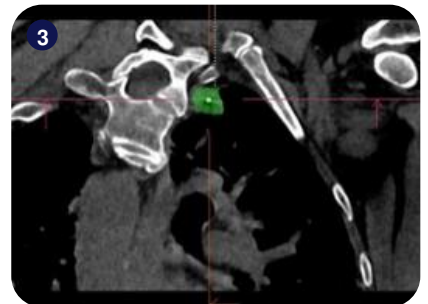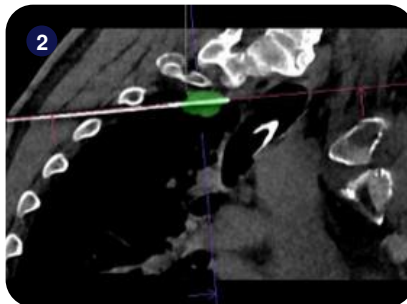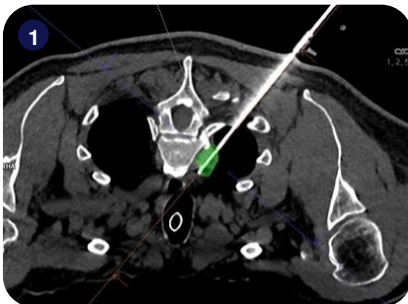

1) Axial view after biopsy needle insertion and tissue harvesting from the right lung apex paravertebral (in green) 2) Sagittal view. 3) Trajectory view accentuating the needle target in the center of the lesion. 4-5) 3D reconstruction of the ribs, vertebrae structures, and inserted guidance needle from caudal view, highlighting the challenging double oblique approach.